

HOW POCKETALKER HELPS

THE POCKETALKER ALLOWS YOU TO AMPLIFY THE SOUNDS AND VOICES MOST IMPORTANT TO YOU. BY PLACING THE MICROPHONE CLOSE TO THE DESIRED SOUNDS, YOU CAN REDUCE DISTRACTING BACKGROUND NOISES. THE POCKETALKER HELPS YOU HEAR WHAT YOU WANT TO HEAR.

HEARING AIDS VS POCKETALKER

A HEARING AID IS A "CUSTOMIZED" MEDICAL DEVICE WHICH AMPLIFIES ALL SOUNDS. THE POCKETALKER PERSONAL AMPLIFIER IS USED TO AMPLIFY SPECIFIC SOUNDS OR VOICES.

CONNECTING YOU TO YOUR WORLD

WHETHER YOU DESIRE TO HEAR THE SOFT VOICE OF A GRANDCHILD, THE WORDS OF A FRIEND OR FAMILY MEMBER, OR THE QUIET SOUNDS OF NATURE, THE POCKETALKER CAN HELP. THE POCKETALKER HELPS KEEP YOU CONNECTED TO THE PEOPLE AND ACTIVITIES MOST IMPORTANT TO YOU.

IF YOU ARE LOOKING FOR AN "OUT-OF-THE-BOX" HEARING ASSISTANCE SOLUTION THAT OFFERS A QUALITY PRODUCT AT AN AFFORDABLE PRICE, THEN THE POCKETALKER IS FOR YOU.



POCKETALKER PRO
A L S O A V A I L A B L E

IDEAL FOR HEALTH CARE
D U R A B L E E A S Y T O U S E R E C H A R G E A B L E



1.800.843.3544 (U.S.A.)
+1.952.943.2252 (OUTSIDE THE U.S.A.)
INFO@WILLIAMSSOUND.COM
WWW.WILLIAMSSOUND.COM



Your Authorized Williams Sound Dealer is:



©2012 Williams Sound, LLC BROM 017E

 **WILLIAMS SOUND®**

HELPING PEOPLE HEAR
P O C K E T A L K E R ® U L T R A



WHERE TO USE

ONE-ON-ONE CONVERSATIONS

RESTAURANTS

RIDING IN THE CAR

SMALL GROUPS

HOW DO I USE MY POCKETALKER® ULTRA?

PLUG YOUR EARPHONE OR HEADPHONE INTO THE HEADPHONE JACK FOR PERSONAL LISTENING.

POSITION THE EXTERNAL MICROPHONE NEAR THE PREFERRED SOUND.

ADJUST THE VOLUME CONTROL TO THE DESIRED LEVEL.

1) Headphone jack

accepts a variety of hearing accessories, including earphones, headphones, and neckloops.

2) External plug-in microphone picks up sound while reducing background noise.

3) Adjustable volume control allows the listener to hear at their comfort level.

